

PAPR Battery and HEPA Filter Instructions

Charging battery with external charging adapter:

1. Remove battery from PAPR unit.
2. Slide external adapter into battery slot.
3. Plug charger into external adapter.



Plug charger in here



Changing battery and assembling correctly before use:

1. Make sure that battery is snapped in place before putting HEPA filter back into unit.
2. When placing HEPA filter in PAPR unit make sure that the arrows on filter are pointing toward the battery and motor.
3. Snap unit cover back on.
4. Check air flow to make sure that PAPR unit was correctly assembled.

